

# Entamoeba Rapid Test Kit

## Instructions For Use

### PRODUCT NAME

Entamoeba Rapid Test Kit

### PACKAGE SPECIFICATION

25 tests/kit

### INTENDED USE

Entamoeba Rapid Test Kit is an in vitro qualitative immunochromatographic assay for the rapid detection of Entamoeba in stool samples. The test results are intended to aid in the diagnosis of Entamoeba infection.

### SUMMARY AND PRINCIPLES OF THE PROCEDURE

Entamoeba histolytica, E. dispar, and E. moshkovskii are morphologically identical but biochemically and genetically different. These parasites colonize the human gut, but only E. histolytica is thought to be capable of causing disease. Amoebiasis is the infection of the human gastrointestinal tract by Entamoeba histolytica, a protozoan parasite that is capable of invading the intestinal mucosa and may spread to other organs, mainly the liver. The disease may manifest itself as an acute, chronic or as an asymptomatic infection. The leading symptom is dysentery whose main consequence one galling of the colon, diarrhoea and abdominal pain. Complications of the disease may be hepatic abscesses, pulmonary abscesses or even cerebral abscesses which, if untreated, usually end in death. Infection occurs by water pollution, vegetables, raw fruits and other food or poorly cooked or washed with infective cysts from contaminated faeces. It is possible that flies and cockroaches carrying cysts from faeces to food Amoebic dysentery occurs frequently in tropical countries but are also cases in temperate and cold areas. In Africa, tropical Asia and Latin America, more than two thirds of the population has these intestinal parasites, although most can be virtually asymptomatic infections.

Entamoeba Rapid Test Kit is a sandwich solid phase immunochromatographic assay. To perform the test, an aliquot of diluted stool sample is added to the sample well of the test cassette. The sample flows through a pad containing antibodies against Entamoeba coupled to red-colored colloidal gold. If the sample contains Entamoeba antigen, the antigen will bind to the antibody coated on the colloidal gold particles to form antigen-antibody-gold complexes. These complexes move on the nitrocellulose membrane by capillary action toward the test line region on which Entamoeba specific antibodies are immobilized separately. As the complexes reach the test line, they will bind to the antibody corresponding to the Entamoeba on the membrane to form of a line. A red control line will always appear in the result window to indicate that the test has been correctly performed and the test device functions properly. If Entamoeba is not present or lower than the detection limit of the test, only the control line will be visible. If the control line dose not developed, the test is invalid.

### KIT COMPONENTS

Each kit contains:

1. Test devices: 25 pieces test devices individually pouched.
2. Sample collection tubes: 25 pieces tubes and 1.0 ml collection solution in each tube.
3. Instructions For Use: 1 piece attached.

### MATERIALS REQUIRED BUT NOT PROVIDED

- Timer or stopwatch.
- Biohazard disposal waste container.
- Disposable gloves and/or protective clothing.

### WARNINGS

1. Read the Instructions For Use completely before using the product. The instructions must be followed carefully as not doing so may result in inaccurate results.
2. The kit is for diagnostic use only.
3. Perform test at room temperature.

### PRECAUTIONS

1. The kit is for professional use only.
2. The Instructions For Use instructions must be followed to ensure optimum test performance.
3. The kit is intended for in vitro diagnostic use.
4. As with all screening assays, any results should be considered presumptive until confirmatory assays have been performed according to local practice or WHO guidelines.

### Safety Precautions

1. Standard precautions for handling infectious agents should be observed when using this kit.
2. Wear protective clothing such as lab coat, safety glasses and disposable gloves when handling specimens and assay reagents.
3. Wash hands thoroughly after use.
4. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

### Bio safety Precautions

Appropriate bio safety practices should be used when handling specimens and reagents. These precautions include, but are not limited to the following:

1. Do not smoke, eat, drink, apply cosmetics or handle contact lenses in areas in which specimens are handled.
2. Dispose of all specimens, used devices and tubes as though they are capable of transmitting infection. The preferred methods of disposal are by autoclave at 121°C for a minimum of 60 minutes or by incineration. Disposable materials may be incinerated. Liquid waste may be mixed with appropriate chemical disinfectants. A solution of 10% bleach is recommended. Allow 60 minutes for effective decontamination. NOTE: Do not autoclave solutions containing bleach.
3. When disposing of sample collection tubes, avoid contact with acid to prevent liberation of a toxic gas.
4. All spills should be wiped thoroughly using a suitable disinfectant such as a sodium hypochlorite solution.
5. Use a separate tube and device for each specimen tested.

### Handling Precautions

1. Do not use if the kit box safety seal is absent, damaged or broken.
2. Do not use any device if the pouches have been perforated.
3. Each device is for single use only.
4. Do not mix sample collection tubes/test devices from different kit lots.
5. Do not use the kit past the expiration date (this date is printed on the kit box).
6. Adequate lighting is required to read the test results.
7. The result should be read immediately after the end of the 10 minutes incubation time following the addition of collected specimen. Do not read results beyond 15 minutes.


### STORAGE INSTRUCTIONS

1. The kit should be stored between 2-30°C and the shelf life is 24 months.
2. The kit components are stable until the expiration date printed on the outer label, when stored as directed. The kit expiry date is determined by whichever of the components has the shortest expiry date. The kit expiry date is not impacted once the extraction Solution has been opened. Do not use kit components beyond overall kit expiry date.
3. If stored refrigerated, ensure that the pouched device is brought to room temperature before opening.
4. Do not freeze the kit.

### TEST PROCEDURE

Allow the test device, specimen, extraction solution to equilibrate to room temperature (15-30°C) prior to testing.

1. Loosen the sample collection tube, take out the sampling rod, insert the feces sample, pick up 50 mg fecal sample (equivalent to the size of match head) and then put the sampling rod back into the tube, and rotate it tighten and shake well.
2. Place the test cassette on a clean and level surface. Break the tip of the sample collection tube, abandon the first two drops and draw two drops (approximately 90ul) into the sample well then start the timer. Avoid trapping air bubbles in the sample well.
3. Wait for the colored line(s) to appear. The test result should be read at 10 minutes. Do not interpret the result after 15 minutes.

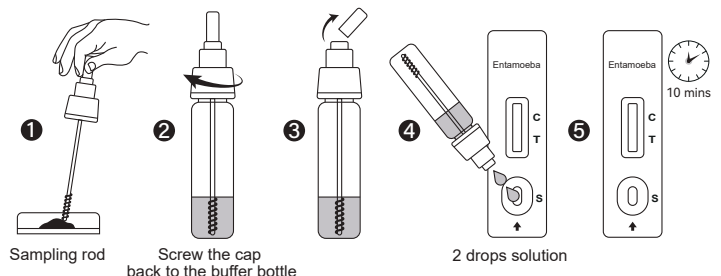


### INTERPRETATION OF RESULTS

**Negative result:** if there is only a quality control line C, the detection line is colorless, indicating that Entamoeba antigen has not been detected and the result is negative.

**Positive result:** Two lines appear on test strip. One colored line should be in the control line region (C), and one colored line should appear in test line region. The color intensities of the lines do not have to match. The result is positive for Entamoeba antigen infection.

**Invalid result:** if the quality control line C is not observed, it will be invalid regardless of whether there is detection line (as shown in the figure above), and the test shall be conducted again.

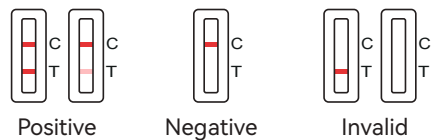

INSTRUCTION APPROVAL AND REVISION DATE

Revision Date: 2025.10.22  
Approval Date: 2025.10.22  
Date of Issue: 2025.10.22

LIMITATIONS

- 1. The kit is for professional in vitro diagnostic use only. The test should be used for the detection of Entamoeba in human feces samples. Neither the quantitative value nor the rate of increase in Entamoeba can be determined by this qualitative test.
- 2. As with all diagnostic tests, all results must be interpreted together with other clinical information available to the physician.
- 3. A negative result may be obtained if the concentration of the Entamoeba present in the specimen is not adequate or is below the detectable level of the test.
- 4. The accuracy of the test depends on the quality of the sample, false negatives may result from improper sample collection or storage.
- 5. After one week of infection, the number of parasites in faeces is decreasing, making the sample less reactive. Stool samples should be collected within one week of the onset of symptoms.

PERFORMANCE CHARACTERISTICS

**Sensitivity and Specificity**  
Clinical study was performed to compare the results obtained by the kit and qPCR. The results indicated that The kit has a high sensitivity and specificity as summarized below:

Entamoeba clinical study		qPCR		Total Results
Entamoeba Rapid Test Kit	Results	Positive	Negative	
	Positive	20	2	22
	Negative	1	214	215
Total Results		21	216	237

Clinical sensitivity=95.24% (95%CI \* 76.18 - 99.88%)  
Clinical specificity=99.07% (95%CI \* 96.7 - 99.89%)  
Accuracy=98.73% (95%CI \* 96.35% to 99.74%)

**Cross Reaction**  
An evaluation was performed to determine the cross reactivity of Entamoeba rapid test; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:

Campylobacter jejuni Campylobacter coli Clostridium difficile Escherichia coli O157:H7 Cryptosporidium parvum Entamoeba dispar	Helicobacter pylori Listeria monocytogenes Salmonella enteritidis Salmonella paratyphi Salmonella typhi Giardia lamblia	Shigella boydii Shigella dysenteriae Shigella flexneri Shigella sonnei Staphylococcus aureus Salmonella typhimurium
---	--	--

INDEX OF SYBOML

	In vitro diagnostic medical device		single-use, Please don't reuse it
	Use-by date		Consult instructions for use
	Cautions		Manufacturer
	Temperature limit		Batch code
	Date of manufacture		Keep Dry
	Avoid overexposure to the sun		Don't use the product when the package is damaged
	Biological risks		

Hangzhou Frenovo Biotech Co., Ltd.  
Address: Room 401,Building 26, Room 401&301,Building 27,No.488-1,Donghu North Road, Donghu Street,LinPing District, Hangzhou City,Zhejiang Province.China.  
Tel: +86-571-89170657  
Email: business@frenovo.com

